


Fetal Fibronectin Specimen

Clinician collection procedure guide


Collect specimen prior to digital examination or manipulation of the cervix to avoid sample contamination.

Collection kits are **FREE!**

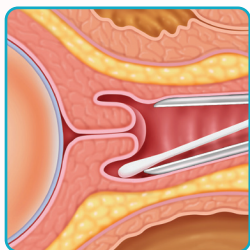


1

During speculum examination, lightly rotate swab across posterior fornix of the vagina for 10 seconds to absorb cervicovaginal secretions.

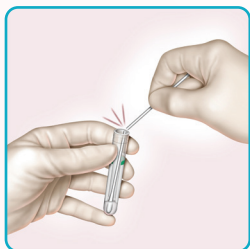

2

Remove swab and immerse polyester tip in buffer; break shaft at score even with top of tube.

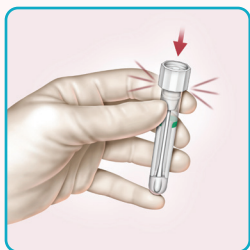

3

Align the shaft with hole inside the tube cap and push down tightly over shaft, sealing tube; ensure shaft is aligned to avoid leakage. **Send fetal fibronectin sample to a lab near you.**

To receive **FREE** Specimen Collection Kits:
Website: www.fFNTTest.com/HCP
Email: CustomerSupport@Hologic.com
Phone: 1-800-442-9892



RapidfFN®

For accurate results, be sure to follow these instructions and see package insert.

Gestational age collection window:

- Symptomatic patients:
24⁰ weeks - 34⁶ weeks
- Asymptomatic patients:
22⁰ weeks - 30⁶ weeks

Specimens should be collected prior to:

- Digital cervical exam
- Collection of culture specimens
- Vaginal probe ultrasound exams

Do not contaminate swab or specimen with:

- Lubricants
- Disinfectants
- Soaps
- Creams

Do not test if patient has:

- Moderate or gross vaginal bleeding
- Advanced cervical dilation (3 centimeters or greater)
- Rupture of membranes
- Suspected or known placental abruption or placenta previa

If patient has had sex in the prior 24 hours:

- A negative result is still valid.
- A positive result may not be valid and should be confirmed >24 hours.

Don't wonder if sending her home is the right decision...

Be confident that it is.

Hologic provides this collection procedure guide as a general informational tool only; it is not an affirmative instruction or guarantee of performance. It is the sole responsibility of medical professionals to read and understand the appropriate package insert and comply with applicable local, state and federal rules and regulations.

MISC-00775-001 Rev. 010 ©2023 Hologic, Inc. All rights reserved. Hologic, Rapid fFN and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshow, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to diagnostic.solutions@hologic.com.