

EASY TO ORDER

## FREE Specimen Collection Kits

Website: [www.fNTest.com/HCP](http://www.fNTest.com/HCP)  
Email: [CustomerSupport@Hologic.com](mailto:CustomerSupport@Hologic.com)  
Phone: 1-800-442-9892



Please contact your local representative for more info and resources.

# RapidfFN<sup>®</sup>

To learn more, visit [fNtest.com/HCP](http://fNtest.com/HCP)

**HOLOGIC<sup>®</sup>**



**25**  
Swabs

Per Box

**18**  
Months

Expiration  
Date

**8**  
Hours

at Room  
Temperature\*

# RapidfFN<sup>®</sup>


\* 3 days refrigerated.

# fFN Specimen Collection Procedure


For accurate results, ensure you follow these instructions and see package insert.

Collection kits  
are **FREE!**


- ▶ During speculum examination, lightly rotate swab across posterior fornix of vagina for 10 seconds to absorb cervicovaginal secretions.

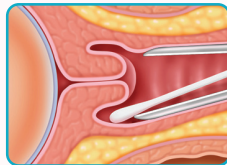

- ▶ Remove swab and immerse polyester tip in buffer; break shaft at score even with top of tube.

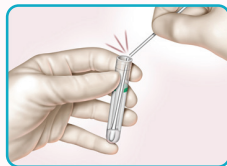

- ▶ Align the shaft with hole inside the tube cap and push down tightly over shaft, sealing tube; ensure shaft is aligned to avoid leakage.

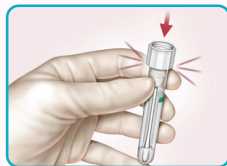

## Gestational age collection window:

- Symptomatic patients: 24<sup>0</sup> weeks - 34<sup>6</sup> weeks
- Asymptomatic patients: 22<sup>0</sup> weeks - 30<sup>6</sup> weeks

## Specimens should be collected prior to:

- Digital cervical exam
- Collection of culture specimens
- Vaginal probe ultrasound exams

## Do not contaminate swab or specimen with:

- Lubricants
- Soaps
- Disinfectants
- Creams

## Do not test if patient has:

- Moderate or gross vaginal bleeding
- Advanced cervical dilation (3 centimeters or greater)
- Rupture of membranes
- Suspected or known placental abruption or placenta previa

## If patient has had sex in the prior 24 hours:

- A negative result is still valid.
- A positive result may not be valid and should be confirmed > 24 hours.

Hologic provides this collection procedure guide as a general informational tool only; it is not an affirmative instruction or guarantee of performance. It is the sole responsibility of medical professionals to read and understand the appropriate package insert and comply with applicable local, state and federal rules and regulations.

MISC-04957-001 Rev. 009 © 2023 Hologic, Inc. All rights reserved. Hologic, Rapid fFN and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to [diagnostic.solutions@hologic.com](mailto:diagnostic.solutions@hologic.com).

RapidfFN®